




Schrader Conversion
Retro-Fit Manual v2.1

LEGENDS:

- 1- O-CPBF-XX-L-GAS (Front Latch Asmby)
- 2- H-MPF-0612 (#6-32 x 3/4")
- 3- O-TRIM-M500C (Chrome Faceplate 5.00"W)
O-TRIM-M325C (Chrome Faceplate 3.25"W)
- 4- O-CON-FIL-PB500 (Filler Frame 5.00"W)
O-CON-FIL-PB362575 (Filler Frame 3-5/8"W)
- 5- C-H-NUT-KEPS (#6-32 Hex KEPS Nut)
- 6- O-X-BAK-RIVET (1/8" x 3/16" GRIP)
- 7- S-DIS-ORING-CO2 (#12 O-ring)
- 8- O-X-SHDR-AD5824 (Schrader Adapter 5/8-24)*
- 9- O-X-CPB-CHKVLV (Except VAC & WAGD)*
- 10- O-X-BAK-MC-PLT (To be GAS Specific)
- 11- H-MTH-08NC-20 (#8-32 x 2" LG. Truss Head)
- 12- H-ORING-111VTN (#111 O-RING)

**NOTES:

- O-CSHR-20-L-GAS
- O-CSHR-27-L-GAS

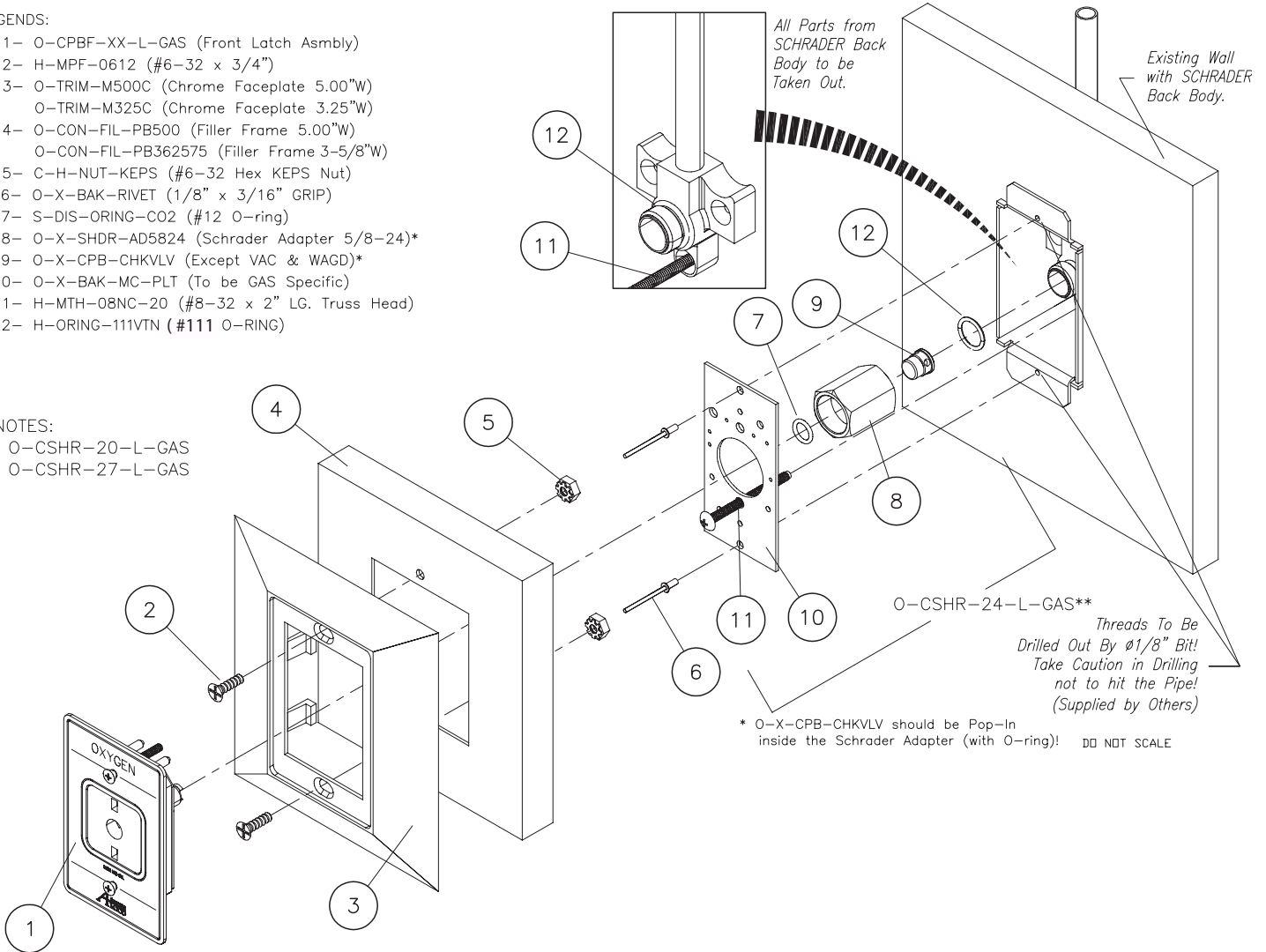

SCHRADER CONVERSION OUTLETS

- O-SHXX-QD-L-G-W (Ohmeda)
- O-SHXX-DI-L-G-W (DISS)
- Not Available in Chemetron
- O-SHXX-PB-L-G-W (Puritan Bennett)

XX defines Thread Size

- 5/8-20UN-2B-RH =20
- 5/8-24UN-2B-RH =24
- 5/8-27UN-2B-RH =27

L defines the Language:

- English (NFPA) = U
- English (CSA/ISO) = E
- French (CSA/ISO) = F
- Spanish (NFPA) = S

G defines the Gas Type:

- Oxygen = O
- Medical Air = A
- Nitrous Oxide = 2
- Carbon Dioxide* = C
- Nitrogen** = N
- MedVac = V
- Waste Anesthetic Gas Disposal = W (NFPA)
- Anesthetic Gas Scavenging System = AG (CSA/ISO)

W defines the Frame Width:

- 3 = 3-5/8"
- 5 = 5.0"

* Only Available in DISS or QD

** Only Available in DISS



85 Fulton Way, Richmond Hill
Ontario, L4B 2N4, Canada

31 Sprague Avenue, Amityville
NY 11701, USA

Toll Free Tel: 1.877.462.6426
Toll Free Fax: 1.866.440.4986
Tel: 905.764.0800 | Fax: 905.764.0862
www.amico.com

represented by: