


Amico Corporation Alert One upgrade to Alert Two

Turn power off at the circuit breaker.



Open front panel by un-screwing two phillips screws.

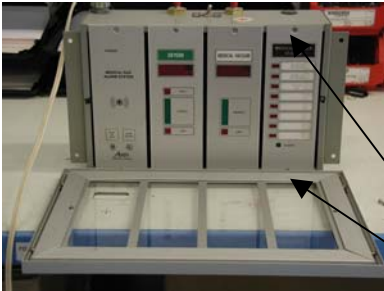


Remove circuit modules by removing screws and ribbon cables.




Remove power supply cover by removing nut.



Remove transformer by removing nuts.

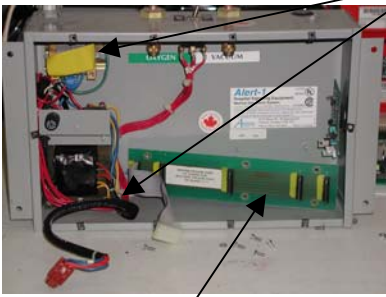


Remove green motherboard.

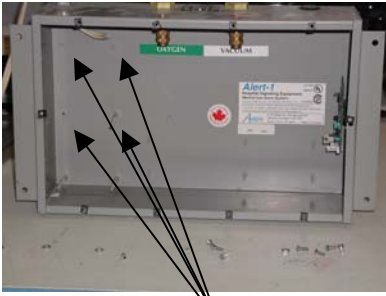


Break off these four studs from box, do not break bottom two studs.



Insert power supply by matching standoff from the alarm box to the power supply.

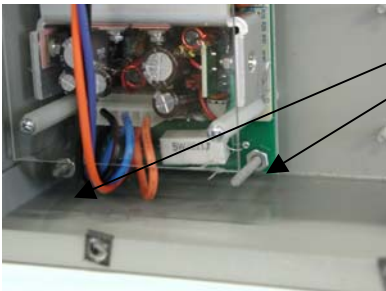

Connect ground wire from power supply to the alarm box.

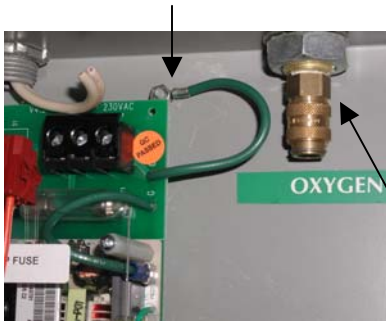

Ensure the pressure of the gas is off and then remove check valve.



Insert demand check to pipe stub using appropriate Oxygen thread sealer.




Connect sensor to demand check valve.



Repeat steps on this page for all other gases.

Remove front frame by removing the screws.

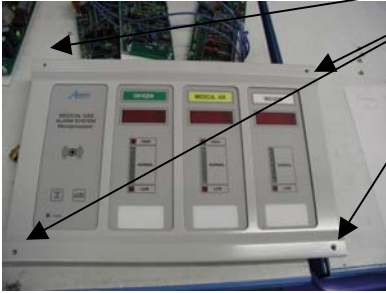

Attach divider plate hinge to the bottom of the alarm box and remove the two 1-inch screws from the top of the frame.




Attach front frame wire.



Close front panel and mark two holes for the front frame.



Drill two holes for a #6 screws into the alarm box and insert two speed nuts.

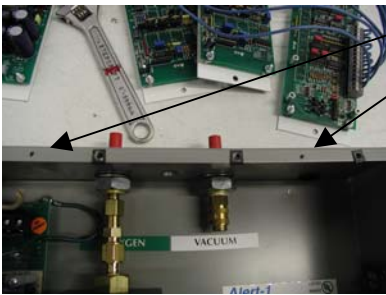

Switch the power supply on, close the front divider plate and insert two 1-inch screws into the alarm box.



Replace the front frame of the alarm panel. Turn the power on at the circuit breaker and ensure the gas supply is flowing.