

Installation Instructions

Hummingbird Power System Driver



Driver Installation

Important, Please Read Carefully

Thank you for your purchase with Amico Accessories Inc.

Our software installation attempts to install drivers automatically, however, some systems need the driver to be set manually. Please see instructions to manually set the PC with the right drivers for our power system.

Host PC Software


The licensed Host software (Graphics User Interface –GUI) is provided by Amico Accessories Inc. for customer use. The PC software is easy to install from the provided software package. The installation file will prompt the user through typical Windows setup instructions.

It is critical that the appropriate driver is loaded for the product. If the driver is not loaded appropriately the software will not function. The driver is part of the software provided and can be found in the programs folder. Typically, the location is C:\Program Files (x86)\AM Series 401.

If for any reason the software is required to be reloaded, it is strongly advised that all software is fully removed from the system, inclusive of drivers, and all components are reloaded.


Setting the Right Driver, Step by Step:

1. In windows, open device manager, see SERIES 401/451 issue

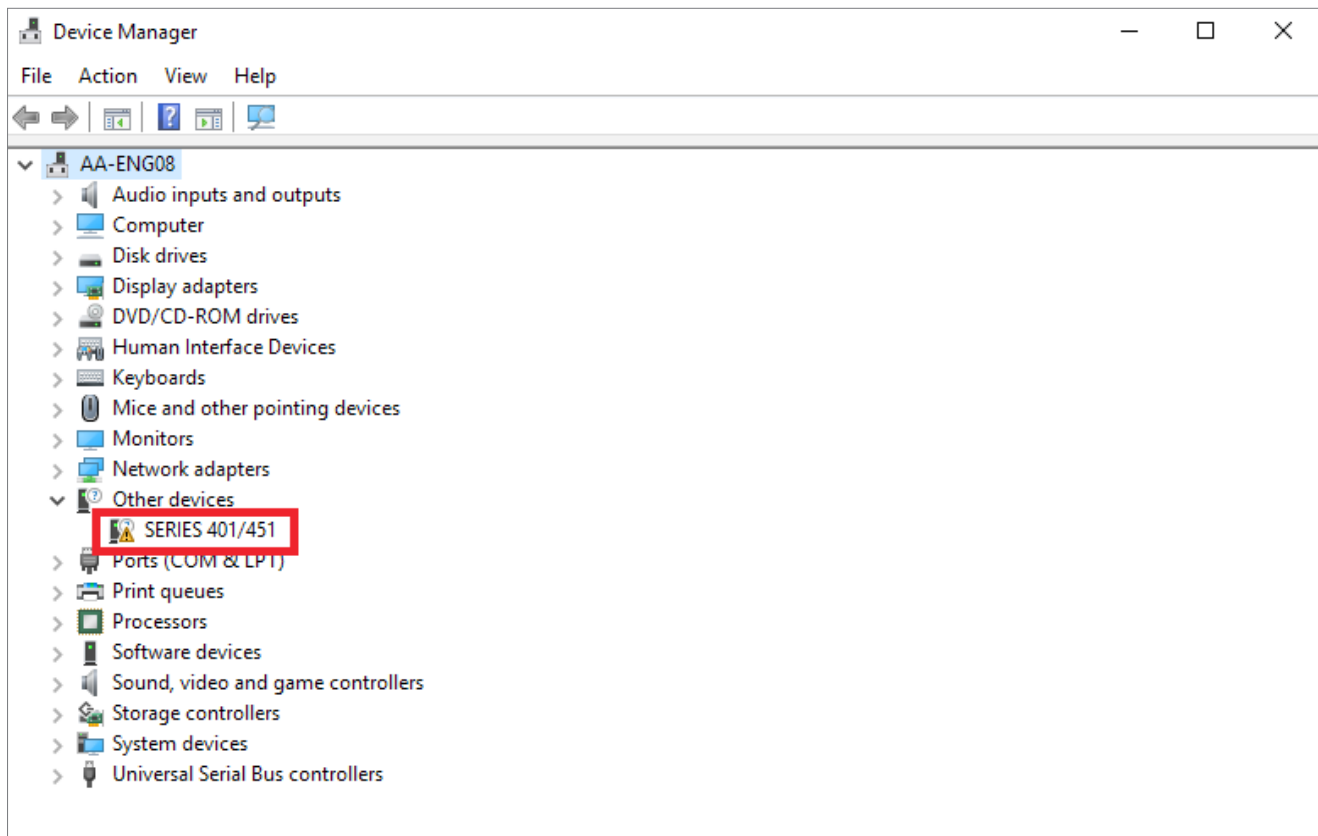


Driver Installation

2. Right click and select Update Driver

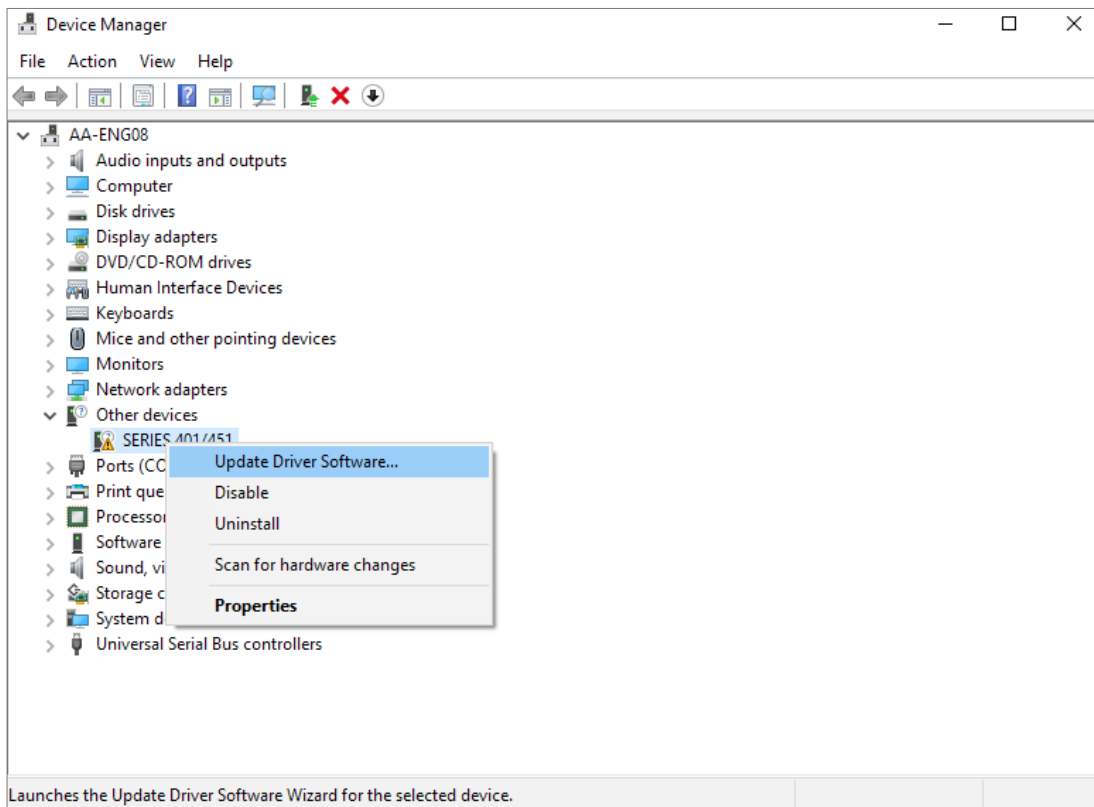


3. Click Browse

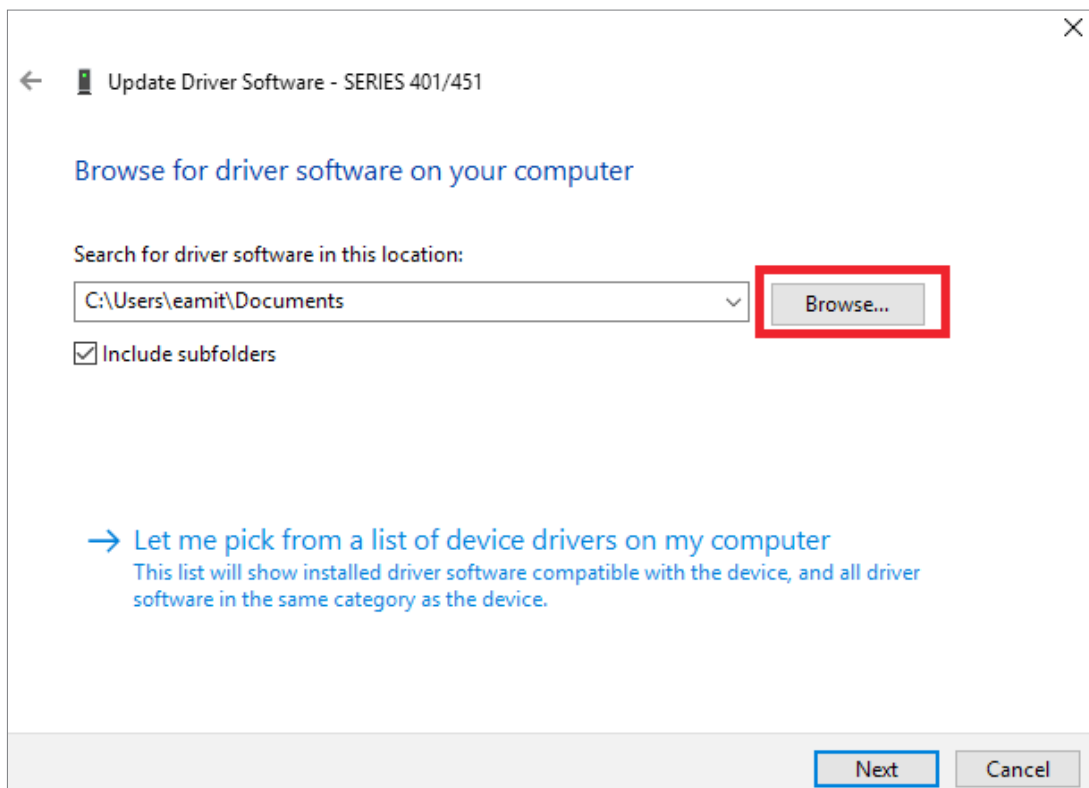


Driver Installation

4. Find the program folder (typically C:\Program Files (x86)\AM Series 401) and click OK

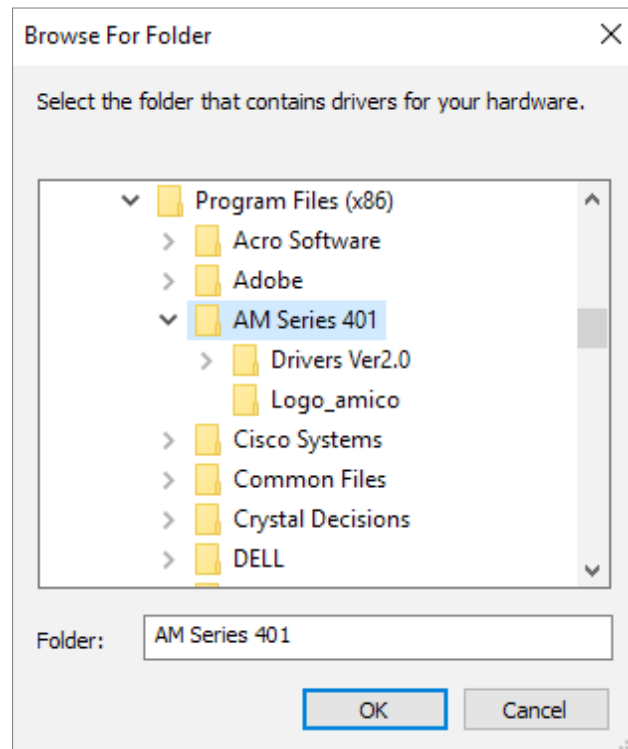


5. Click Next

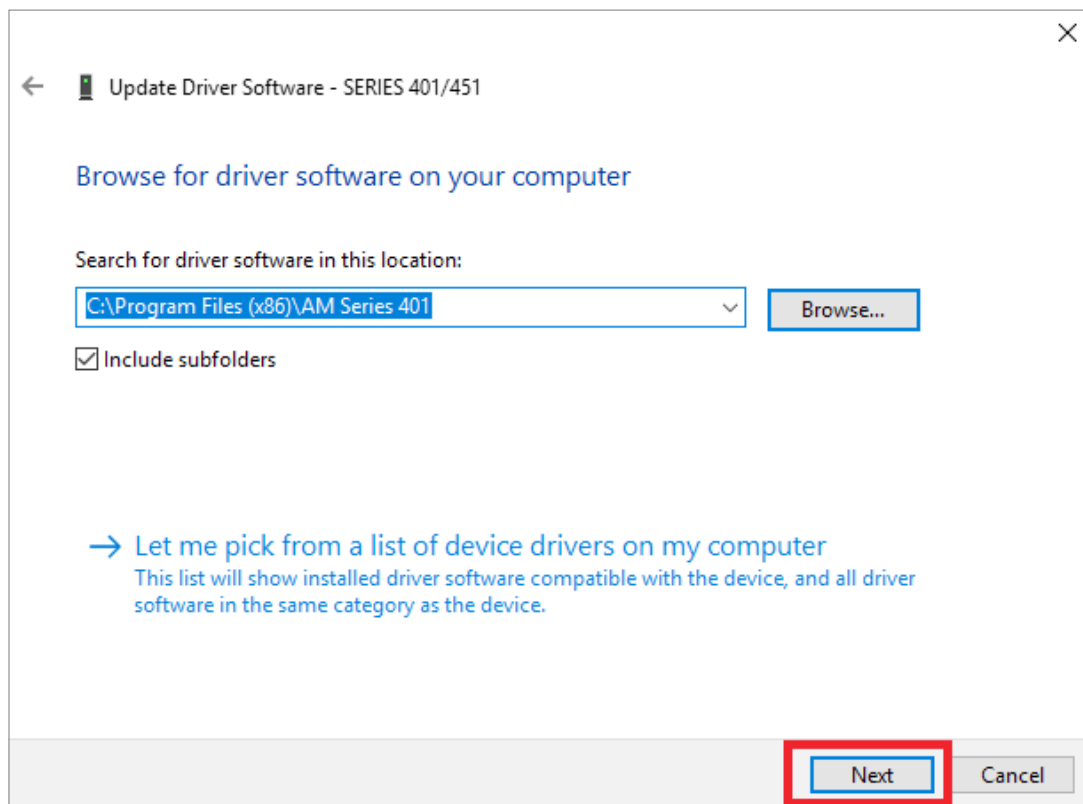

Driver Installation

6. Success! Click Close

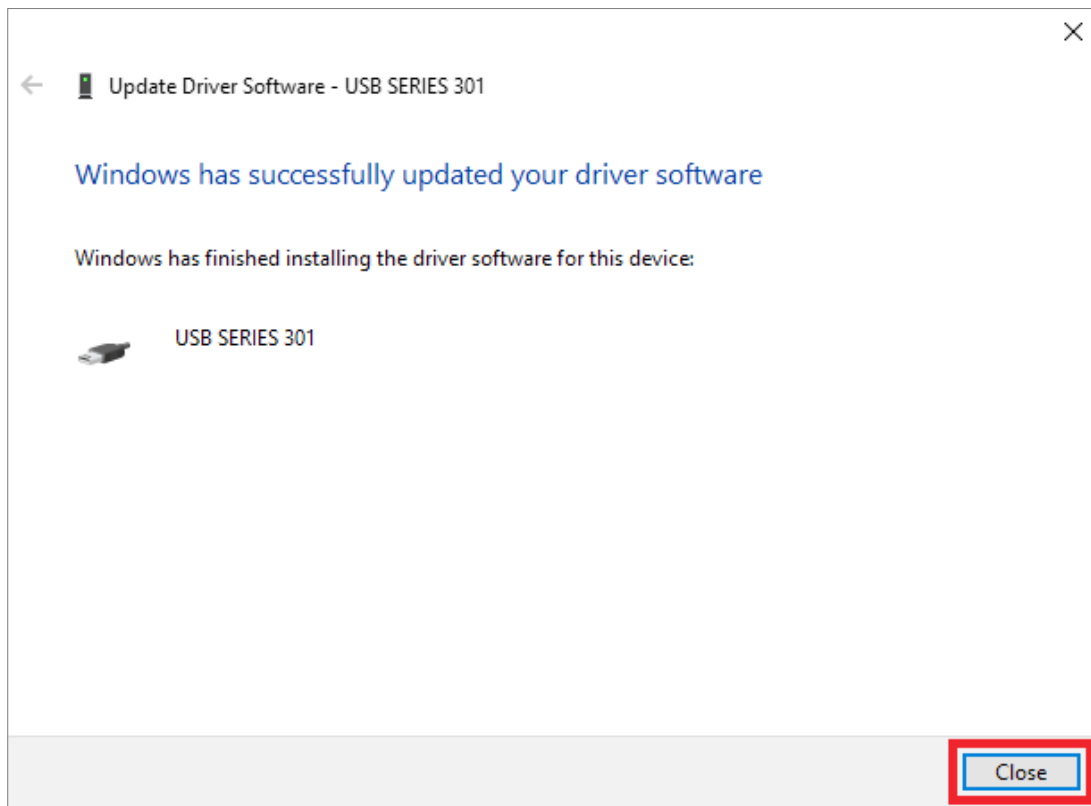



Warranty Policy - Equipment Mounting Solutions

Amico Accessories Inc. warrants all mounting accessories to be free from defects in material and workmanship for a period of twelve (12) months from the date of shipment. Within this period Amico Accessories Inc. will repair or replace any part which is proven to be defective.

Amico Accessories Inc. will warrant its materials to be free from defect for an additional period of four (4) years, (five [5] years from the date of shipment). Within this period, Amico Accessories Inc. will replace any part which is proven to be defective, at no charge. Shipping and Installation costs after the first twelve (12) months will be borne by the Customer.

This warranty is valid only when the product has been properly installed according to Amico Accessories Inc. specifications, used in a normal manner and serviced according to factory recommendations. It does not cover products that are not manufactured by Amico Accessories Inc. It does not cover failures due to damage which occurs in shipments or failures which result from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to acts of God.

Amico Accessories Inc. shall not be liable for incidental or consequential damages resulting from use of equipment.

All claims for warranty must first be approved by Amico Accessories Inc. A valid Return Goods Authorization (RGA) number must be obtained from Amico Accessories Inc. prior to commencement of any service work. Warranty work, which has not been pre-authorized by Amico Accessories Inc., will not be reimbursed.

AMICO ACCESSORIES INC. DOES NOT HONOR VERBAL STATEMENTS CONCERNING THE WARRANTY.

The distributor and/or dealer are not sanctioned to create verbal warranties about the product described in this agreement. Any statements will not be honored or be made part of the agreement of sale. This document is the final complete and exclusive terms of the agreement.

THIS WARRANTY IS INCLUSIVE AND REPLACES ALL OTHER WARRANTIES.

Amico Accessories Inc. shall not, under any circumstances be liable for incidental or consequential damages including, but not limited to, profit, loss of sales or injuries to person(s) or property. Correction of noncompliance as noted above will result in completion of all liabilities of Amico Accessories Inc., whether based on agreement, neglect or changed materials, designs or specifications without notice.

All claims for warranty must first be approved by Amico Accessories Inc. Service Department: accessories@amico.com or 1.877.264.2697. A valid Return Goods Authorization number must be obtained from Amico Accessories Inc. prior to commencement of any warranty claim.

www.amico.com

Amico Accessories Inc. | 85 Fulton Way, Richmond Hill, ON L4B 2N4, Canada
71 East Industry Court, Deer Park, NY 11729, USA
Toll Free Tel: 1.877.264.2697 | Tel: 905.763.7778 | Fax: 905.763.8587
Email: info@amico.com | www.amico.com