

Majestic Series Single Tier Horizontal Headwalls

General Specifications

The Amico Majestic Series Single Tier Headwall shall be manufactured by Amico Corporation, in accordance with job specific shop drawings and documents. The complete Headwall assembly shall be cULus listed. The following is a general specification, and components listed may not be present or required in final product.

The Majestic Series Headwall system is listed by Underwriters Laboratories Inc.(UL) in Canada and the United States of America. This product also complies with the seismic requirements of, and is approved by the State of California's Office of Statewide Health Planning and Development (OSPHD). Amico Corporation manufactures all products in accordance with the National Fire Protection Association (NFPA), NFPA-99c (current edition) and National Electric Code (NEC).

The installation contractor is responsible for compliance with all local, state, and federal codes applicable to the installation of medical gas and electrical systems.



Submittals and Verifications

Job specific shop drawings shall be produced for each project. These shop drawings will clearly indicate the area of Medical Gas termination, and electrical connection points inside the Headwall System. As Built drawings will be produced, and emailed at the time of shipment.

Amico Corporation will manufacture equipment as per signed approval drawings and verifications documents, which are provided in the job specific submittal package.

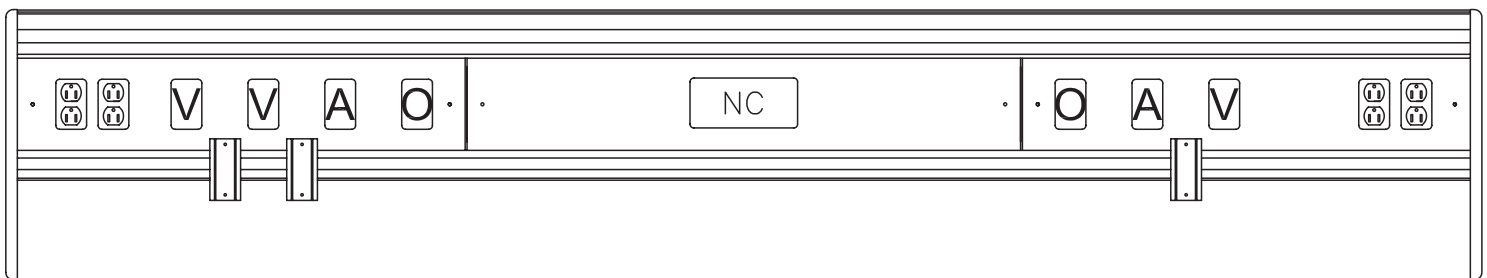


Figure 1. Majestic Series Single Tier Headwall System

Majestic Series Single Tier Horizontal Headwalls

Materials and Construction

Enclosure shall be constructed of extruded, anodized aluminum alloy sections to provide a modular surface mounted unit with integrated accessory rails for equipment management.

The Majestic system can be manufactured in lengths that range from 12" (305 mm) to 120" (3048 mm). Units that require a length beyond 120" (3048 mm) will be divided into sections using painted connection plates. On-site medical gas and electrical connections between sections will be required.

Each headwall system has integrated raceways and a single area connection for electrical wiring for each type of power (critical, normal and low voltage/communication).

All med gas piping shall be hard-piped and brazed to a single point of connection.

Fascia panels, top and bottom cover panels shall be removable for access for easy installation and maintenance of headwall services.

Integrated Accessory Rails

The Single Tier Headwall System has two (2) accessory channels, which are integrated into the aluminum extrusion assembly. The accessory channels are clear etched anodized.

Laminated Fascia

Aluminum strips with High pressure laminate (HPL), color and pattern selection by customer.

Covers and End Caps

Top and bottom cover panels are made of power coated extruded aluminum. End caps are made of injection molded ABS Fire Retardant plastic.

Service Chase (Optional)

Shall be constructed of 6063-T5 extruded aluminum, with passage space for conduit and medical gas piping. High Pressure Laminate MDF inserts shall finish the exposed front, and shall match the laminate color selected.

Medical Gas Outlets

Amico Corporation manufactures several compatible connection types for Medical Gas and Vacuum Outlets. The outlet connection type shall be as called for on the signed approval documents.

Medical Gas Manifold

Medical gas distribution shall be supplied by pipe drops via the service chase or single point termination zone as indicated on the drawing. All Medical Gas Outlets and piping shall be brazed and tested in accordance with NFPA 99c. All Medical Gas piping shall be Type L copper pipe. Each outlet, piping and manifold shall pass a 24 hour standing pressure test.

Electrical Wiring

Wire for standard and critical branch power circuits shall be #10 or #12 (as specified) type THHN stranded copper wire, 600 volt, with heat resistant thermoplastic insulation for hot (black) and neutral (white). Grounds shall be #10 type RTHHN stranded copper wire (green). All ground conductors shall be installed in conduit.

Grounding and Bonding

All ground conductors shall be installed in conduit. Each power receptacle shall have a ground conductor connected to a grounding screw. A grounding bus will be installed in each device barrier box at rough in locations, and shall be installed as to insure grounding for the complete power system.

Electrical Receptacles

Shall be Hospital Grade 15 or 20 amp, 120 or 277 volt, UL listed and marked Hospital Grade. Quantity and type as shown on drawings. Amico shall furnish and install receptacles with wiring required, as shown on submittal shop drawings. Electrical contractor shall insure compatibility of plug on accessory equipment to be used with these devices:

Duplex Receptacles shall be NEMA style 5-15R or 5-20R, color Ivory for use on normal power circuits, and color Red for use on critical branch power circuits.

Simplex Receptacles shall be NEMA style 5-15R or 5-20R, color Ivory for use on normal power circuits, and color Red for use on critical branch power circuits.

Safety Receptacles (if required) shall be duplex type, be NEMA style 5-15R or 5-20R, color Ivory for use on normal power circuits, and color Red for use on critical branch power circuits. Receptacles shall limit proper access to energized contracts and shall accept both 2 wire and 3 wire plugs.

Locking Receptacles (if required) shall be simplex type, 20 amp, 120 or 277 volt, color black, and be HUBBELL lock 23000 HG style, or equivalent.

Majestic Series Single Tier Horizontal Headwalls

Low Voltage Data Provisions

Shall be connected to device junction box via conduit or raceway. Amico Corporation shall include pull cord extending from junction box to service provision. Telephone jack, Blank faceplates, Data port or other Low Voltage device shall be furnished, installed and wired by Low Voltage contractor.

Switching

Shall be Industrial Grade 120 or 277 volt, 15 or 20 amps. Switch type options include SPST, 3-Way or Momentary, as shown on submittal shop drawings. Low Voltage Switching will be 0-12 volts, 15 amps unless otherwise noted. Amico Corporation shall furnish, pre-install and wire all switches.

Lighting (Optional)

Direct and Indirect lighting can be provided by the Amico 3' or 4' Overbed lighting system. Two fluorescent fixtures and ballasts shall be located in the Overbed light enclosure. These fixtures provide lighting on and above the patient bed. Each fixture shall accommodate four (4) fluorescent bulbs (supplied by others).

Installation

Mounting and final connections are the responsibility of the installing contractor. This includes receiving, storage, erection, overhead bracing, clean up, touch up, and carton disposal. The contractor shall supply all necessary installation materials, including such items as tools, fasteners, caulking and electrical lamps.

The electrical contractor shall be responsible for the final electrical hook-up at service connection locations, as well as interconnection wiring on multi-sectional units. Once the installation is complete, the electrical contractor shall test equipment function and high voltage services / grounding in accordance with local electrical code requirements. It is the responsibility of the hospital and/or the approval authority to notify Amico if Local electrical code varies or differs from that of NEC. Amico manufactures all products in accordance with NEC (current edition).

The medical gas contractor shall be responsible for piping and final connection of all medical gas services. Amico Corporation will manifold medical gas plumbing internal to the Headwall System, and provide single point of connection as per the submittal documents. The medical gas contractor shall be responsible for purging, pressure testing, gas identification and system certification in accordance with NFPA 99c (current edition) or CSA-Z7396.01-06.

Please refer to the Single Tier Majestic installation instructions for further detail.

Optional Equipment

Amico Accessories

Shall be provided in the quantities and types as ordered. Accessories shall be installed by others, in the locations determined by the end user. Please see the Amico Accessories catalog for additional product and mounting information.

Product Code Definition

HWXX-01T-P-MYYY

- X = Variable Chase Width (00, 12", 14", 19")
- P = Private or Semi-Private (P, S)
- YYY = Length in Inches (example – 096)

Distributed By: