# Instructions for Use

## DH-W35 Ophthalmoscope Series

### Manuel d'instructions

### DH-W35 Série Ophthalmoscope

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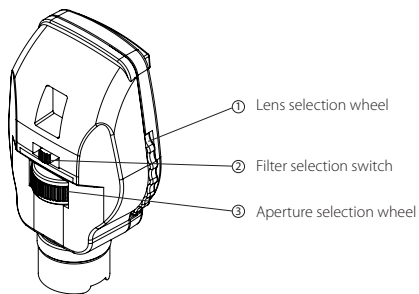
### Ophthalmoscope serie DH-W35

the same instrument or any other ophthalmic instrument using a visible light source during the previous 24 hours. This will apply particularly if the eye has been exposed to retinal photography.

- CAUTION:** Federal law restricts this device to sale by or order of a physician.
- CAUTION:** This product should not be immersed in fluids.

### Connection to Handles

**To connect:** Slide the ophthalmoscope down onto the top of the handle. Twist the instrument head clockwise.

**To disconnect:** Twist the Ophthalmoscope counter-clockwise and pull.

**Anti-theft Lock:** After the ophthalmoscope is attached to the handle, use the Hex key provided to tighten the set screw located on the neck of the ophthalmoscope head. Turn until the set screw stops, **DO NOT** overtighten. The ophthalmoscope is now locked to the handle. To remove the ophthalmoscope, loosen set screw until the head can be removed.

### Bulb Replacement

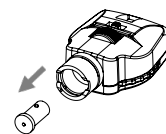
- CAUTION:** Bulb may be hot. Make sure the bulb is cool before removing.

Slide out bulb from bottom of Ophthalmoscope head. Replace with new bulb making sure index tab is aligned. Push inward until firmly seated.

### Cleaning Instructions


**Note:** Excess solution entering the optical assembly could damage internal components. Use caution to ensure cloth is not saturated with solution.

Disinfect housing using soft cloth moistened with alcohol. Windows may be cleaned with a cotton swab or lens paper dipped in alcohol.

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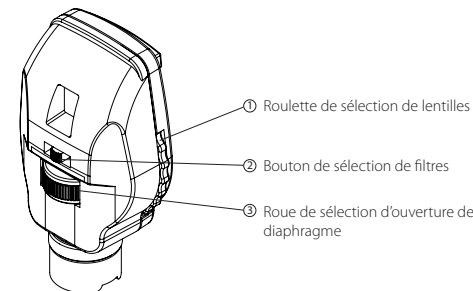## English

### Warnings and Cautions

- CAUTION:** To minimize lamp housing temperatures, on time should not exceed 2 minutes with off-time not less than 10 minutes.
- CAUTION:** Because prolonged intense light exposure can damage the retina, the use of the device for ocular examination should not be unnecessarily prolonged, and the brightness setting should not exceed what is needed to provide clear visualization of the target structures. This device should be used with filters that eliminate UV radiation (<400 nm) and, whenever possible, filters that eliminate short-wavelength blue light (<420 nm).

The time to reach a potential optical radiation hazard for this device is 3.5 minutes. The retinal exposure dose for a photochemical hazard is a product of the radiance and the exposure time. If the value of radiance were reduced in half, twice the time would be needed to reach the maximum exposure limit.

While no acute optical radiation hazards have been identified for direct or indirect ophthalmoscopes, it is recommended that the intensity of light directed into the patient's eye be limited to the minimum level which is necessary for diagnosis. Infants, aphakes and person with diseased eyes will be at greater risk. The risk may also be increased if the person being examined has had any exposure with

### Apertures

- Large Aperture:** Standard aperture for dilated pupil and general examination of the eye.
- Medium Aperture:** Provides easier access to undilated pupil.
- Small Aperture:** Allows entry into very small, undilated pupils.
- Slit Aperture:** Used primarily to determine retinal elevations and depressions
- Fixation Aperture:** Used for assessment of the degree and direction of eccentric fixation.
- Cobalt blue filter:** Used in conjunction with fluorescein dye for the detection and examination of corneal scars and abrasions.

### Filters

- Polarizing Filter:** Used to eliminate corneal glare and reflection.
- Red Free Filter:** Used to examine blood vessels in fine detail. This green filter blocks red rays, showing blood vessels as black against a dark green background.

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