

Standard Fetal Monitor Cart

Product Specifications and Operating Manual

IMPORTANT: SAFETY INSTRUCTIONS

Read and understand all of the safety and operating instructions contained in this document. If you do not understand these instructions, or have any questions, contact your supervisor, dealer or the manufacturer before attempting to use the cart.

⚠ WARNING: Failure to comply with these instructions could result in injury to the patient and/or healthcare provider and/or damage to the cart.

User Responsibility

- This product is to be used only by people who have been trained to properly operate it
- **DO NOT** open more than one drawer at a time
- **DO NOT** lean on flip top surface, pull-out trays or drawers
- **DO NOT** exceed a weight of 20 lbs evenly distributed on flip top surfaces
- **DO NOT** exceed a weight of 10 lbs per drawer and 5 lbs on keyboard trays
- **DO NOT** exceed a weight of 30 lbs on fetal monitor trays
- **DO NOT** use on an inclined plane
- Be aware of all possible pinch points

Intended Use

Do not exceed weight restrictions for drawers, pull-out trays or flip top surfaces. Flip top surfaces are intended to be used as a temporary additional surface space.

General Maintenance


ATTENTION: Service of this cart should only be performed by properly trained individuals.

Periodic review of this cart is recommended. If any damage or defects are present, the cart should not be used. This includes parts that may have been altered, are worn or missing or become contaminated. If any of the above are noted, immediate repair or replacement is required.

If this product is subject to improper maintenance, repair, use and/or abuse leading to malfunction of the device, replacement is the sole responsibility of the user.

Standard Specifications:

- Overall height from floor to top work surface 41" (+/- 1")
- Work surface: (W x D) 24" x 25 1/4" (+/- 1")
- 4" casters – 2 locking and 2 non-locking
- Customization available

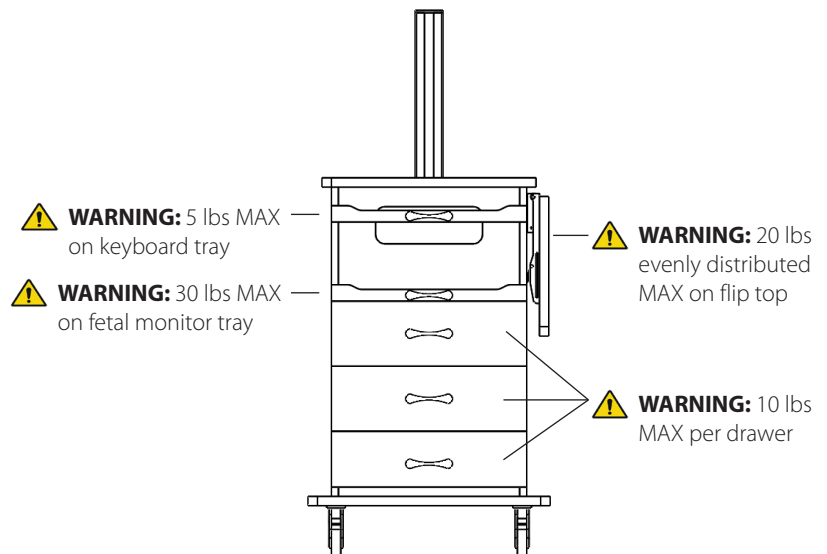

Standard Fetal Monitor Cart

Directions for Cart Use

⚠ WARNING: Failure to comply with these instructions could result in injury to the patient and/or healthcare provider and/or damage to the cart.

Before Moving the Cart

- Make sure all drawers and/or doors are completely closed
- Make sure the flip top surface is in the down position
- Make sure the keyboard and fetal monitor trays are not extended
- Make sure the monitors are not extended outside the work surface of the cart
- Make sure any additional accessories are properly fastened to the cart
- Ensure that the locking casters are unlocked
- Take all possible care to make sure path of travel is free from obstruction

Moving the Cart

- **DO NOT** move the cart by pushing/pulling on the keyboard tray
- **DO NOT** move the cart by pushing/pulling on the fetal monitor tray
- **DO NOT** move the cart by pushing/pulling on the flip top surface
- **DO NOT** move the cart by pushing/pulling on the monitor mounting post and/or monitors
- **DO NOT** move the cart at a high rate of speed, use caution while moving it
- **DO NOT** take corners at a high rate of speed
- Keep both hands on the cart while in motion
- Make sure feet are clear of the cart while in motion and path is free from obstruction

Using the Cart Features

- Ensure that the front locking casters are locked before using the drawers, flip top surface or pull-out trays
- Ensure there is sufficient clearance around the perimeter of the cart before engaging the flip top surfaces, pull-out trays and/or drawers
- **DO NOT** overload or pry on any accessories
- **DO NOT** lean against fetal monitor while fetal monitor tray is extended

Cleaning Instructions

- Amico Patient Care recommends using a mild cleaning agent or detergent to clean this product
- Dampen a soft cloth with diluted cleaner and wipe the surface to remove dirt and fingerprints
- Wipe the surface with a soft dry cloth to remove any moisture