

Sidekick with RFID Reader

Quick Installation and Setup Instructions



Parts Supplied



RFID USB Dongle



RFID/WAVE ID Sticker

Installation Tools (Optional)



Small Screwdriver



Needle Nose Pliers

Instructions

1. Locate the dual surface designated to be RFID upgraded.

NOTE: It has to have a textured top surface and capacitive switch to be upgradable.



2. Clear loose items off the top and then fold up the top surface.



3. Locate the RFID cover on the bottom of the gable on your right.



4. Remove the RFID cover.






Optionally, use a small screwdriver to gently pry it open.



Sidekick with RFID Reader

Quick Installation and Setup Instructions



<p>5. Plug the USB dongle into the USB socket.</p> 	<p>6. Optionally, use needle nose pliers to hold the USB dongle.</p> 
<p>7. Ensure the USB dongle is plugged in properly.</p> 	<p>8. Put the cover back in place.</p> 
<p>9. Make sure the cover is secured.</p> 	<p>10. Flip the dual surface back down and place the RFID/WAVE ID sticker to indicate RFID reader location.</p> <p>Plug in and test.</p> 